

EQUIPMENT INSTALLATION GUIDE
DS2278



DS2278






DS2278 scanner configuration procedure

1. ChronoMEDIC Production must be closed.
2. Connect the gray wire to the base.
3. Plug the base into the USB port on the computer.
4. Wait for the computer to detect it (about 10 seconds).
5. Place the scanner on the base, a series of different sound signals will be heard.
6. When the light on the scanner flashes green, scan the following bar codes, another series of different sound signals will be heard.



Restore Defaults

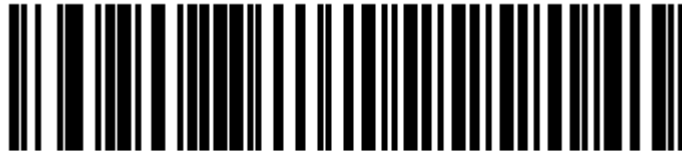

IBM Hand-Held USB

7. Wait for the computer to detect it.
8. Open ChronoMEDIC Production, normally a message for adding the device should appear, otherwise:
 - a. Go to the Utilities tab.
 - b. CONFIGURATION button, then USB button.
 - c. Click on the new device and scan your employee card, it should appear in the square at the bottom.
 - d. Go to the Consultation tab.
 - e. HISTORY (tray or product).
 - f. Scan your employee card.
 - i. Check that an error message is displayed (ex.: Product xxx doesn't exist).