

Configuring the EY-H2

1. Close ChronoMEDIC Production.
2. Plug the reader wire into the handle and then into the computer.
3. Scan the first USB COM image.



4. Scan the second Add CR+LF image.



5. Open ChronoMEDIC production:
 - In the Utilities / Configuration Buttons / RS-232 Button

On the right side of the screen:

- Scanner: present
- Model: Select STANDARD
- Prefix: leave blank.
- Serial port: try the ones that are available (in black).
- Speed: 4,800
- Parity: none
- Bit: 1 (data): 8
- Flow control: none
- End character 13&10

6. Scan a code if you see the answer in the box at the bottom RS-232 Test and it displays the correct identification, it's done.